BELLMAN AUDIO DOMINO

- DE
- DK
- EN
- ES
- FR
- IT
- JP
- NL
- PL
- SE
- SI-CN
- TR-CN
- APP

PERSONAL HEARING SYSTEM

Inhalt

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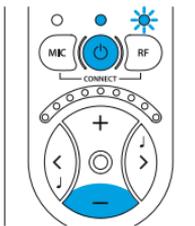
Appendix

Settings


Bellman Audio Domino has more settings than those you can access in user mode. By going into the Settings Menu you can adjust settings that you do not normally need to change or which are adjusted when first used and never need to be changed again.

Settings menu

Start the Receiver Unit with the On/Off button  (6).

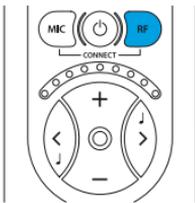Hold down the On/Off button  (6) and press the Volume Button  (10) within 1 second until the RF LED (4) starts to blink. When the RF LED (4) starts to blink, you can release both the buttons.

Press the **RF** button (7) within 2 seconds of the moment the RF LED (4) starts to blink and hold down the button until the RF LED (4) goes out. Release the **RF** button (7) immediately when the RF LED (4) has gone out.

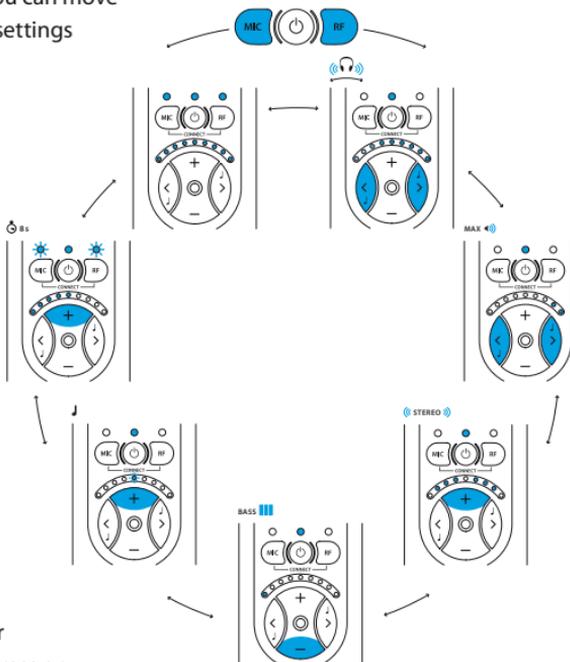
You are now in Balance Control mode.

Overview of the Settings Menu

In the settings menu you can move between the different settings

by pressing the **RF** button (7) or the **MIC** button (5).


Each press on the **RF** button moves you clockwise on to the next setting in the list and the **MIC** button takes you anti-clockwise to the previous setting in the list.

When you have finished adjusting your settings, give a short press on the On/Off button  (6) to return to normal user mode.

Balance control

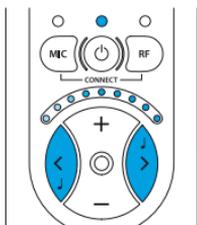The balance between the right and left-hand channels can be set to compensate, for example, for a hearing loss in one ear.


Press the < button (11) to adjust the balance on the left and the > button (12) to adjust the balance on the right.

The standard setting is centred balance.

Basic attenuation

Different headphones have different sensitivity levels and you can compensate for this with basic attenuation. What you actually set is the attenuation you require at the maximum power output you want to have (which is therefore 0 dB attenuation). This settings is useful when you want to adjust the maximum volume in user mode for a pair of headphones or in-ear phones with a higher sensitivity than the BE9122 Bellman Stereo Headphones or, for example, if you want to limit the volume for a child.

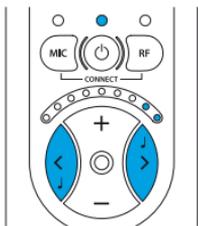
MAX 

Use the **<** button (11) to reduce the basic attenuation (increase the volume) and the **>** button to increase the basic attenuation (reduce the volume). The basic attenuation is adjusted in increments of 3 dB.

The standard setting is 0 dB attenuation, which corresponds to the maximum output signal.

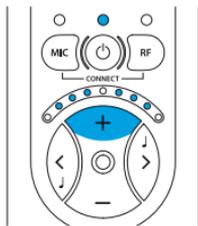 STEREO 


Stereo/Mono

The line-in signal can be rendered in either Stereo or in Mono. The speech intelligibility can be increased in certain listening situations such as when watching TV if the signal is in Mono.

Press the **+** button to select Stereo.

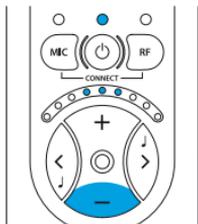MONO 


Press the **-** button to select Mono.

The standard setting is Stereo.

Bass reduction

Low-frequency interference such as fans and road noise can be tiring if you are exposed to it for a long period. By activating bass reduction, low-frequency interference and sounds are reduced.

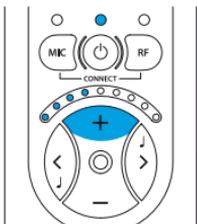BASS 

Press the **+** button (9) to activate Bass Reduction.

BASS 

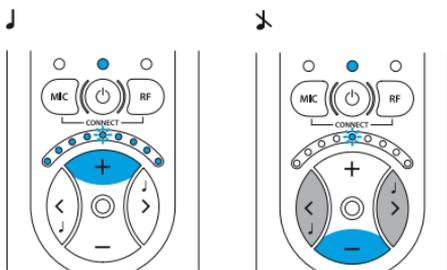
Press the **-** button (10) to deactivate Bass Reduction and get level frequency response.

The standard setting is deactivated Bass Reduction.

Adjusting the Tone Control


The Tone Control Blocking function has been developed in case you want to make it easier for the user to use Bellman Audio Domino. This setting can be used to choose whether the user should be able to adjust the tone control or not. If you choose to switch off the tone control buttons, you can preset the tone control that will be used in user mode.

Activating and deactivating Tone Control in User Mode

Press the **+** button (9) to activate the tone control buttons and the **-** button (10) to deactivate them.

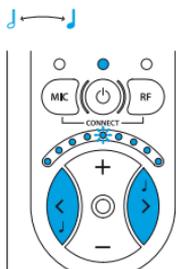The standard setting is tone control activated.


Presetting the Tone Control

Press the **<** button (11) and the **>** button (12) to preset the tone control to the level that will be used in user mode.

The standard setting is Tone Control centred (level frequency response)

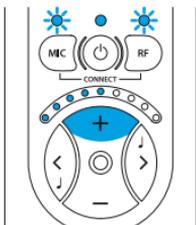
Please note that if the tone control is on, the preset value will be changed as soon as the Tone Control is changed in user mode.

Resetting to the factory setting

You can use this to reset all settings to the standard settings that the products were supplied with from the factory.

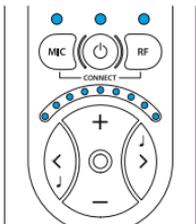Press and hold down the **+** button (9) until all the LEDs on the display are lit. After resetting is completed, the display will blink for a short period.


Version information

The product version information is presented here via a pattern of lit and blinking LEDs. This internal information is only intended as an aid for servicing and support.

Pairing of units

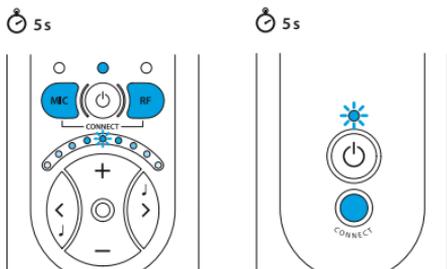The Transmitter and Receiver Units in the Bellman Audio Domino system must have the radio link established in order for radio communication to operate. To make it easier to get started, a Receiver Unit and a Transmitter Unit which are paired together are always supplied so that they will work together. Where you want to pair a Receiver Unit with a Transmitter Unit which is different from the Receiver Unit it was originally supplied with, you can do this in the Settings Menu. Proceed as follows to pair up the units:

- Start both the units with the On/Off buttons (6, 18).
- Go into the Settings Menu.
- Hold down the **CONNECT** button (19) on the Transmitter Unit.
- Hold down both the **MIC** button (5) and the **RF** button (7) on the Receiver Unit.

A "running light" on the Receiver Unit's Volume and Tone LED Indicator (8) moving towards the centre indicates that the link has been established.

Please note! If you pair up a Transmitter Unit and a Receiver Unit in the Settings Menu, this replaces the pairing carried out in the factory. This can be useful when changing a Transmitter Unit under guarantee or if you want to pair up several Receiver Units to a Transmitter Unit (e.g. in a conference room).

Testing

It is easy to test the Bellman Audio Domino Personal Hearing System. If Bellman Audio Domino does not operate as described below, you can carry out further troubleshooting as instructed under Troubleshooting.

How to test

The testing of Bellman Audio Domino has been divided into three parts. One part for the Receiver Unit for which only the Receiver Unit is required, one part for the Transmitter Unit for which both units are required and finally one for charging the two units.

Receiver Unit

- Switch on the unit and make sure that the Battery Status LED (3) lights up green.
- Connect the headphones to the Headphone Socket (13) and scrape carefully on the round microphone (1) with your finger and check that there is a scraping sound in the headphones.
- Press the **MIC** button (5) and make sure that the Microphone LED (2) changes colour and lights up orange.
- Scrape carefully on the small microphone hole on the top (1) and the round microphone (1) with your finger and check that there is a scraping sound in the headphones.
- Press the buttons (9, 10, 11, 12) and check that the Volume and Tone Indicator changes.

Transmitter Unit

- Switch on the unit and make sure that the Battery Status LED (17) lights up green.
- Press the **RF** button (7) and make sure that the External Sound LED (4) lights up green.
- Carefully scrape on the round microphone hole (16) on the top of the Transmitter Unit with the finger and check that there is a scraping sound in the headphones.
- Press the **RF** button (7) and make sure that the External Sound LED (4) changes colour and lights up orange.
- Carefully scrape on the round microphone (16) on the Transmitter Unit with the finger and check that there is a scraping sound in the headphones.
- Connect an external sound source to the External Sound Source Input (20) and check that the sound from the sound source is heard in the headphones. The external sound source input (20) can be tested, for example, by connecting a CD player or a radio to the input via the audio cable supplied. The transmitter unit automatically switches to an external sound source when a connector is connected to the External Sound Source Input (20).

Charging

- Connect the charger to the mains socket and check that the LEDs on the charger light up green.
- Connect the charging cables to both the charger and to the charging sockets on the Receiver Unit and the Transmitter Unit (14, 21), see the fold-out section of the manual.
- Check that the LEDs on the charger change from green to red.
- Check that the Battery Status LEDs on both the Transmitter Unit and Receiver Unit light up orange.

Troubleshooting

Problem	Solution
The sound is louder in one ear than in the other.	Adjust the Balance in the Settings Menu.
The sound volume is low even though the volume is set to max.	Adjust the Basic Attenuation in the Settings Menu.
The sound is only in Mono even though it is connected via a stereo cable to a sound source which provides stereo sound.	Change to Stereo in the Settings Menu.
The bass reproduction is weak.	Switch off Bass Reduction in the Settings Menu.
The Tone Control cannot be adjusted.	Use the Settings Menu to activate the Tone Control buttons or to adjust the sound.
The Receiver Unit is behaving strangely.	Reset the unit to the factory settings in the Settings Menu.

Problem	Solution
The Receiver Unit loses its link to the Transmitter Unit when you switch off the Receiver Unit.	Pair up the Receiver Unit and Transmitter unit permanently in the Settings Menu.
Humming occurs when connecting the stereo cable during charging.	Only use the charger supplied, BE9142.

Technical information

Power supply

Battery power	3.7V Li-Ion battery
Operating time	8 hours (standard battery)
Transmitter charging time	2.5 hours (standard battery)
Receiver charging time	2.5 hours (standard battery)

Input signals

Microphone	Built-in microphone (omnidirectional or directional)
External sound source input	2.5 mm stereo jack plug (0 dBV max input signal)

Output signals

Headphone socket	3.5 mm stereo jack plug
Distortion	0.05% THD (electrical)
Frequency range	40-10 000 Hz
Tone control	+/- 10 dB, cut-off frequency 1.6 kHz
Output power	100 mW (16 Ω load)

Radio communication

Frequency	2.4 GHz, 3 channels
Range	25 metres, clear line of sight, indoors. Walls, concrete and metal reduce the range.
The following may also affect the performance:	<p>More than one Receiver is connected to a Transmitter and at least one of the Receivers is too far away from the Transmitter.</p> <p>Too many Receivers are connected to the same Transmitter.</p> <p>Too many transmitters at 2.4 GHz or too many Domino systems close to one another.</p>
Security	Encrypted communication, automatic channel searching, coexists with other radio transmitters on 2.4 GHz

Additional information

For indoor use only.

Dimensions WxHxD	48 x 99 x 22 (max) mm
Weight	BE2210: 69 g, BE2230: 66 g
Colour	Black/dark grey metallic with silver-coloured buttons.

Regulatory requirements

CE, FCC, RoHS, WEEE,

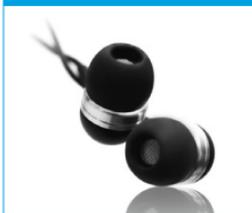
Accessories

The following accessories are available:

- BE9122 Stereo headphones
- BE9124 Ear phones
- BE9125 Stetoclips
- BE9126 Stereo cable set, 5m (16 feet)
- BE9127 External microphone, 5m (16 feet)
- BE9128 Neck loop
- BE9143 USB car charger
- BE1360 Scart switch

EAR PHONES

HEADPHONES

STETOCCLIPS

NECK LOOP

MICROPHONE

These original accessories are especially developed to maximize the performance of your Domino. Contact your nearest dealer or visit: www.bellman.com

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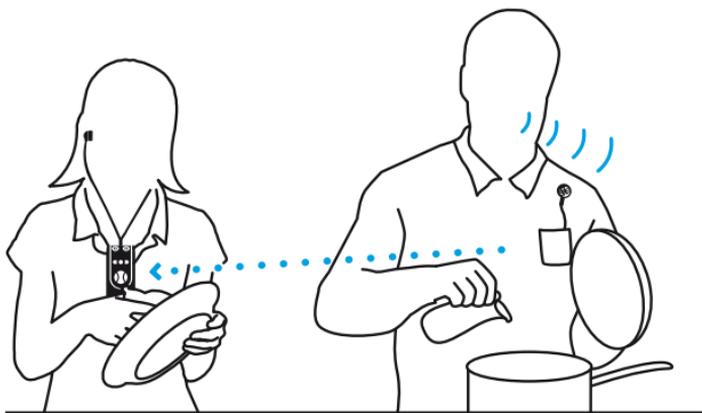
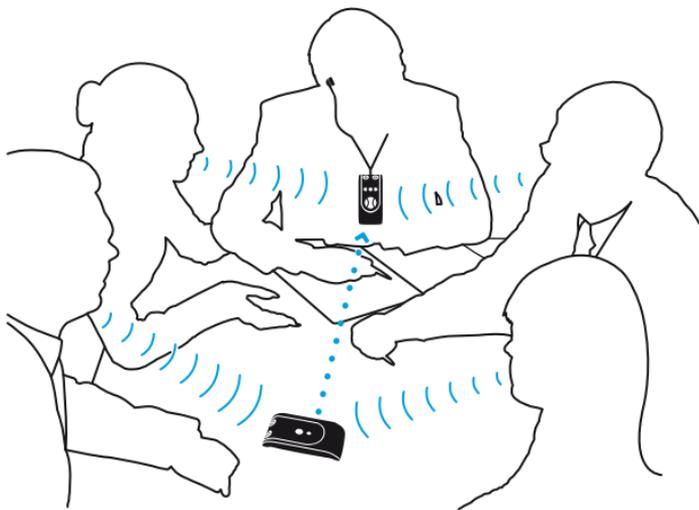
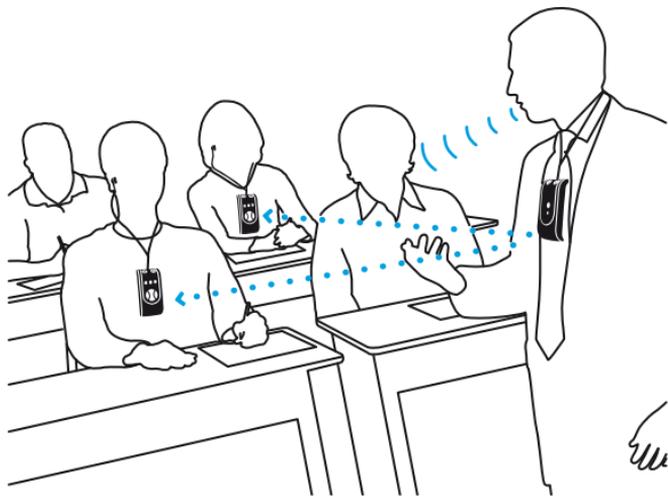
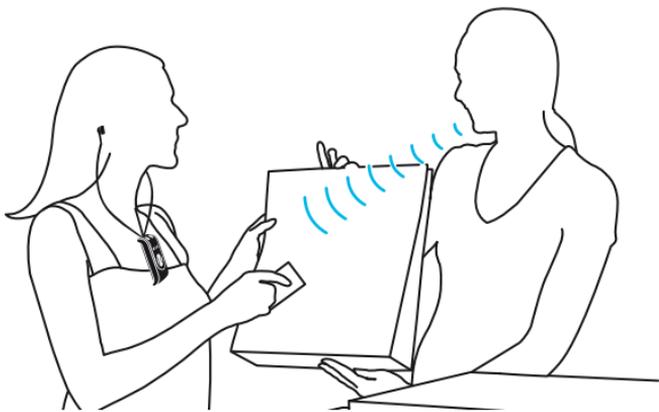
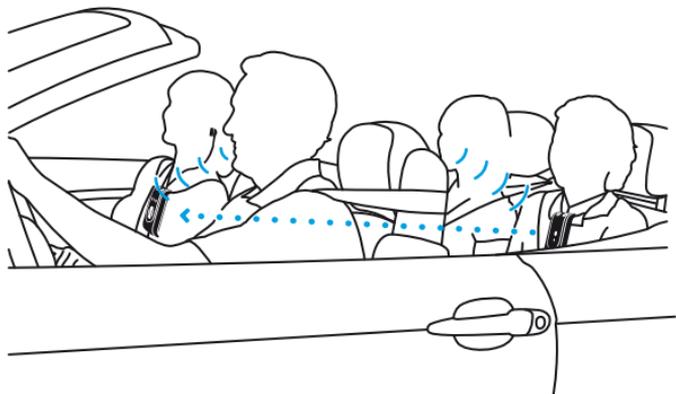
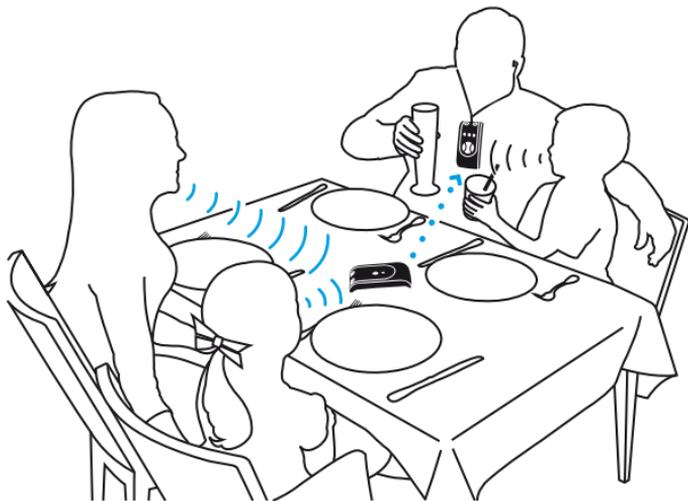
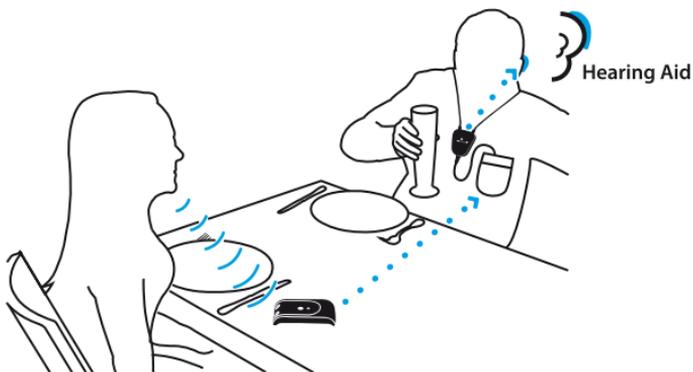
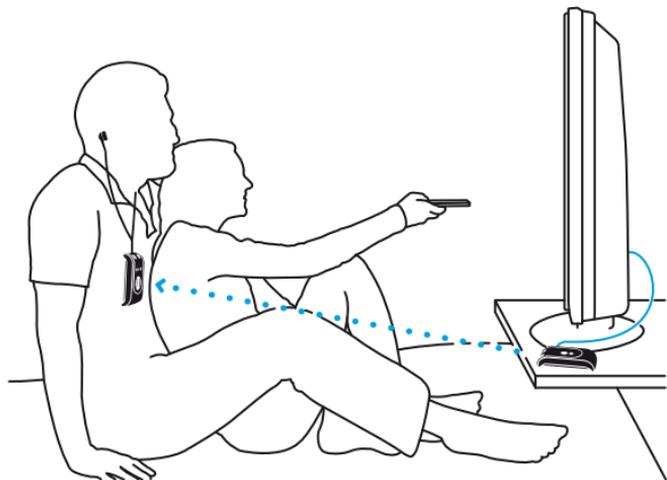
Warranty conditons

Dear customer. Thank you for your purchase of this Bellman & Symfon® product. Our vision is to supply outstanding products that are of great value for our customers. For this reason, we have put all our best efforts into designing and manufacturing this product. However, if you should have any questions about the product or if you require repair, please contact us.

General warranty conditions. Your Bellman & Symfon® product is covered by a limited warranty. Please read the following conditions carefully and keep this information in a safe place. Please also save your receipt or other proof of purchase as you will need this for warranty claims.

Warranty period. The warranty is depending on the type of product purchased and is valid from date of purchase. Bellman & Symfon offers warranty as per below:

Bellman Products in general	Two years of warranty from date of purchase
Bellman Neckloop	One year of warranty from date of purchase
Rechargeable batteries	One year of warranty from date of purchase
Bellman Accessories	90 days of warranty from date of purchase

Warranty conditions. Bellman & Symfon warrants that the Product that you have purchased is free from manufacturing defects in materials and workmanship when dispatched from our warehouse. Your sales receipt is your proof of the date of purchase and the warranty period commences as of that date. During the warranty period, Bellman & Symfon will have the option for the defective product to be repaired or replaced. The replacement product will be of equal or greater function and in good working and physical order. Such a replacement will be supplied from refurbished stock. The repaired or replaced product will be under warranty for the remainder of the original product's warranty period.

Warranty limitations. This warranty automatically becomes void if the product has been physically damaged or rendered defective (a) as a result of accident, misuse, or abuse, or other circumstances beyond Bellman & Symfon's control; (b) by the use of parts or peripherals not authorized by Bellman & Symfon; (c) as a result of normal wear and tear; (d) by use within an improper operating environment; (e) by modification of the product; (f) by the serial number or product code sticker being removed or defaced; (g) as a result of service rendered by anyone other than a Bellman & Symfon Authorized Service Agent; (h) as a result of the product being operated other than in accordance with Bellman & Symfon AB's users manual. Neither does the warranty cover failure due to causes outside Bellman & Symfon's control such as failure in your phone carrier's line service (if applicable) or Acts of God. No warranty is offered for disposable batteries.

Return policies / warranty claim. Please see www.bellman.com/warranty