Bellman® Visit 868

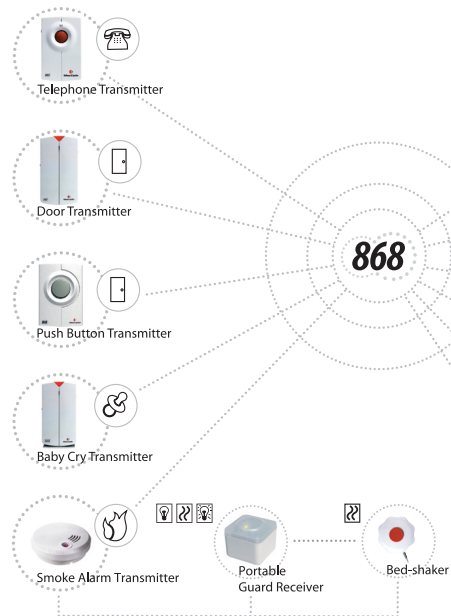
Wrist Receiver & Charger

Model BE1560 & BE1570


Bedienungsanleitung	DE
Brugsvejledning	DK
Instrucciones	ES
Käyttöohje	FI
Mode d'emploi	FR
Instructions	GB
Οδηγίες	GR
Használati utasítás	HU
Manuale d'Uso	IT
Gebruiksaanwijzing	NL
Bruksanvisning	NO
Instrukcja obsługi	PL
Instruções	PT
Bruksanvisning	SE
Appendix	APP

Transmitters + Add-on-solutions

BE 1465 Smoke alarm system for use in all EU countries is in compliance with the essential requirements and other relevant provisions of directives: R&TTE 1999/5/EC, LVD 1973/23/EC, EMC 1989/336/EC, MDD 1993/42/EEC.

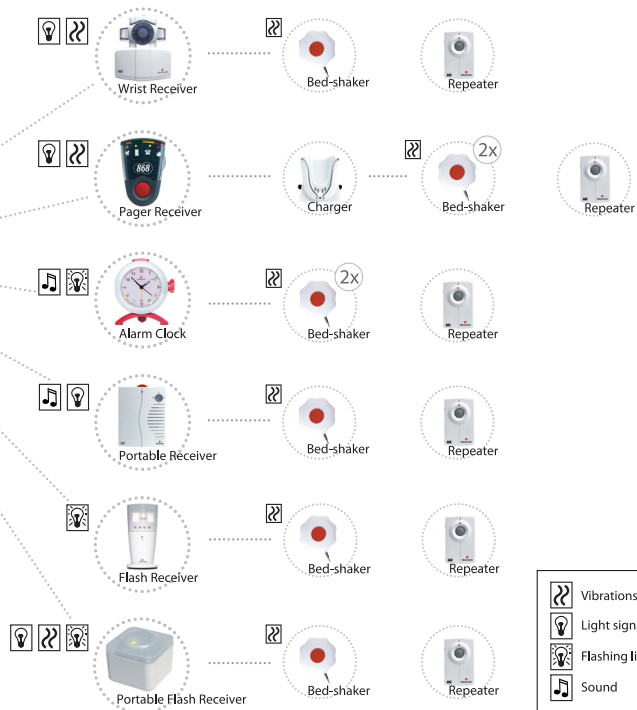Receivers + Add-on-solutions

	Vibrations
	Light signal
	Flashing light
	Sound

Bellman® Visit 868

Wrist Receiver & Charger

Model BE1560 & BE1570

For use in all EU countries

Hereby, Bellman & Symfon AB, declares that this BE1560 Wrist Receiver and BE1570 Wrist Receiver Charger is in compliance with the essential requirements and other relevant provisions of directives: R&TTE 1999/5/EC, LVD 1973/23/EC, EMC 1989/336/EC and MDD 1993/42/EEC.

Correct use and disposal of batteries:

Replace only with the same or equivalent type recommended by the manufacturer. Please dispose of old, defective batteries in an environmentally friendly manner in accordance with the relevant legislation.

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Bellman Visit 868 Wrist Receiver, BE1560 and Bellman Visit Wrist Receiver Charger, BE1570

Introduction

Thank you for choosing products from Bellman & Symfon.

The Bellman Visit 868 System consists of a number of radio transmitters and receivers. The transmitters detect different events in the surrounding area and transmit a radio signal to the receivers. The receivers pick up this signal and provide indications using light, sound and/or vibration.

The transmitter determines what type of light, sound or vibration should be displayed so that the reason for the indication is evident.

Read through the entire user manual first and then start to install the system.

Refer to the illustration of the Bellman Visit 868 System on the inside of the cover.

Getting started

Unpacking, installing and testing the unit

The Charger is usually placed on a level surface and the Wrist Receiver should normally be fastened around the wrist or placed in the Charger.

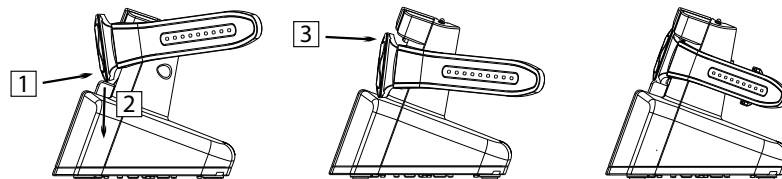
1. Remove the small paper tab sticking out of the battery cover (9) and connect the power supply unit to the mains voltage and to the power supply unit socket  (15). The green LED (10) should come on and stay on as long as the Charger is powered by an external power supply.
2. Place the Wrist Receiver in the Wrist Receiver Charger. The green LED (12) should light up. If the battery is completely flat, the Wrist Receiver starts up but can only be used when placed in the Charger until it has been charged for approximately two hours.

3. Connect a BE1270 Bellman Bed-shaker (accessory) to the socket (14) and press the Charger's Test Button (11). The Bed-shaker should vibrate as long as the button is pressed. Please note that when the Test Button (11) is pressed, the Charger is only powered by the back-up batteries.
4. A Bellman Visit 868 transmitter is required to test the radio reception. Press the test button on the Bellman Visit 868 Transmitter. The Bellman Visit 868 Wrist Receiver will emit a light signal and will vibrate via the connected Bed-shaker. Please note that the Wrist Receiver itself does not vibrate when it is placed in the Charger.
5. The Bellman Visit 868 Wrist Receiver is used around the wrist just like a wristwatch.

The BE1560 Bellman Visit 868 Wrist Receiver is placed in the BE1570 Bellman Visit Wrist Receiver Charger in order to charge it and to connect, via the Charger, a BE1270 Bellman Bed-shaker (accessory) which is placed under the pillow to wake up the user. Charging is controlled by the Wrist Receiver and takes up to 8 hours. During charging, the LED (12) on the Wrist Receiver will light up green.


To charge the Wrist Receiver:

1. Angle the Wrist Receiver slightly outwards so that the 2 small holes in the edge of the Wrist Receiver point downwards and move it towards the lower moveable part of the Charger
2. Press the Wrist Receiver and the moveable button downwards
3. Move the upper part of the Wrist Receiver towards the back edge and release carefully. The green LED (12) should light up.

To use the Wrist Receiver when it has finished charging:

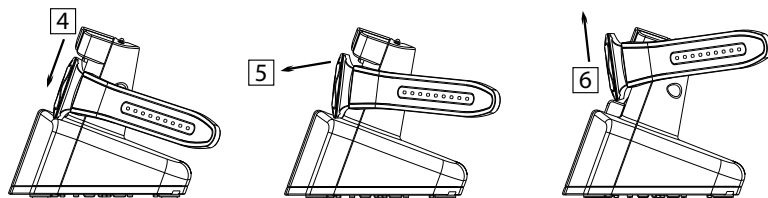4. Press the Wrist Receiver and the moveable button downwards
5. Angle the upper part of the Wrist Receiver
6. Lift the Wrist Receiver out of the Charger

Function

General

The BE1560 Bellman Visit 868 Wrist Receiver is a receiver in the Bellman Visit 868 System and the BE1570 Bellman Visit Wrist Receiver Charger is a charger used to charge the Wrist Receiver's internal battery. Both products are for indoor use only and the Wrist Receiver attracts the attention of the user using light and vibrations. Normally the Wrist Receiver Charger should always be connected to mains power via

the power supply unit, thus automatically charging its internal back-up batteries. This enhances safety since the back-up batteries remain operational even when there is a power failure.

If the Wrist Receiver is connected to the Wrist Receiver Charger, a BE1270 Bellman Bed-Shaker (accessory) can also be connected and will therefore also be alerted to events while the Wrist Receiver is being charged. This combination provides better functionality through the Bed-shaker alarm options when the user is sleeping.

It is activated via radio signals from any transmitter within the Bellman Visit 868 System.

Radio key

On delivery all Bellman Visit 868 units are tuned to the same Radio Key. If you have a neighbour with a similar system, you can change to different Radio Keys so that you do not affect each other's systems.

The Radio Key on this receiver can be changed by holding down the Function Button (1) for about five seconds until the LEDs  (3) and  (4) blink alternately. Release the Function Button (1) and then press the transmitter's test button so that the receiver's LEDs     (2-5) blink to confirm that the Radio Key has been changed. All units in a Bellman Visit 868 System must have the same Radio Key in order to operate as a group. Refer to the user manual for the relevant unit.

Indicators and Signals

It is generally the transmitters in the Bellman Visit 868 System that determine how the receivers will indicate an alarm. See the description in the relevant transmitter user manual for further information.

System indicators

The LEDs     (2 - 5) that indicate which transmitter has activated the Bellman Visit 868 Wrist Receiver normally have the following meanings:

- Orange LED  (2) indicates a baby cry transmitter
- Green LED  (3) indicates a door transmitter
- Yellow LED  (4) indicates a telephone transmitter
- Red LED  (5) indicates a smoke alarm

If the green  (3) and yellow  (4) LEDs blink alternately, this indicates that the Bellman Visit 868 Wrist Receiver is in radio key selection mode. The Wrist Receiver will then wait for a radio signal from a transmitter in the Bellman Visit 868 system which will adjust the receiver to the same Radio Key as the transmitter is tuned to.

Vibration

The built-in vibrator in the Wrist Receiver vibrates differently according to which transmitter has activated the Wrist Receiver. Refer to the user manual for the relevant Bellman Visit 868 transmitter for more information about vibration patterns.

A BE1270 Bellman Bed-shaker (accessory) can be connected to the Charger so that the Wrist Receiver can also be used when sleeping. The Bed-shaker is placed under the pillow and when the Wrist Receiver is placed in the Charger to charge it during the night, the user is woken by the bed-shaker when the Bellman Visit 868 Wrist Receiver is activated.

Please note! The Wrist Receiver will not vibrate whilst inserted in the Wrist Receiver Charger. A BE1270 Bellman Bed-shaker (accessory) must be connected to the Charger to provide a vibratory alert. Other indicators will act as normal.

Apart from a BE1270 Bellman Bed-shaker, it is also possible to connect other products such as the BE1341 Bellman Alarm Clock, the Bellman Visit 868 Guard Receiver and so on to the vibrator output.

Power supply

If the LED  (6) on the Wrist Receiver blinks yellow for a short period, this means that the battery is flat and must be charged. Only use rechargeable VARTA V40H type NiMH batteries for the Wrist Receiver.

If the LED (12) on the Wrist Receiver Charger lights up green, this indicates that the Wrist Receiver is connected to the Charger and that the Wrist Receiver's battery is being charged.

If the LED (10) on the Wrist Receiver Charger lights up green, this indicates that the Charger is connected to the power supply unit and that the Charger is powered by mains voltage.

If the LED (10) on the Wrist Receiver Charger blinks green, this indicates that the Charger is powered by the back-up batteries. Only use rechargeable TMK AAA60CT type NiMH batteries for the Wrist Receiver Charger.

Please note! The Bellman Visit 868 Wrist Receiver and Bellman Visit Wrist Receiver Charger CAN ONLY BE USED WITH RECHARGEABLE NiMH BATTERIES!

Non-rechargeable batteries will start to leak if recharged! The acid in the batteries will damage the electronics in the BE1560 Bellman Visit 868 Wrist Receiver and the BE1570 Wrist Receiver Charger. The resulting damage will not be covered by any guarantee. Only use Varta V40H batteries for the Wrist Receiver and TMK AAA60CT batteries for the Charger.

1. Function button
2. Orange LED 
3. Green LED 
4. Yellow LED 
5. Red LED 
6. Yellow low battery LED indicator 
7. Wrist Receiver battery cover
8. Wrist Strap
9. Charger battery cover
10. Green LED for mains connection
11. Test button
12. Green LED for charging Wrist Receiver
13. Charging area for Wrist Receiver
14. Bed-shaker socket 
15. Power supply unit socket 

Troubleshooting in brief

Problem	Solution
Nothing happens.	• Check that the power supply unit is connected correctly and that the green LED (10) on the Charger lights up. • Charge or renew the battery in the Wrist Receiver. Only use a rechargeable VARTA V40H NiMH battery. If the battery is completely flat, you can still use the Wrist Receiver by placing it in the Charger. The Wrist Receiver needs to be charged for about two hours before it can be used separately. • Check that the Wrist Receiver has been correctly placed in the Wrist Receiver Charger and that the LED (12) lights up green.
It is not possible to charge the Wrist Receiver.	• Check that the power supply unit is connected correctly and that the green LEDs (10 and 12) on the Charger light up. • Charge or change the batteries in the Wrist Receiver Charger. Only use rechargeable TMK AAA60CT NiMH batteries.
Nothing happens when the power supply unit is disconnected. Nor does the Test Button (11) work.	• Charge or renew the batteries in the Charger. Only use rechargeable TMK AAA60CT NiMH batteries.

Problem	Solution
The LED  (6) on the Wrist Receiver blinks yellow!	• Charge or renew the battery in the Wrist Receiver. Only use a rechargeable VARTA V40H NiMH battery. If the battery is completely flat, you can still use the Wrist Receiver by placing it in the Charger. The Wrist Receiver needs to be charged for about two hours before it can be used separately.
The Bed-shaker does not vibrate when the Wrist Receiver is activated.	• Check that the bed-shaker is correctly connected to the Wrist Receiver Charger. • Check that the Wrist Receiver has been correctly inserted in the Wrist Receiver Charger and that the LED (12) lights up green.
The Wrist Receiver is not activated.	• Check the batteries in the transmitters. • Check that the Wrist Receiver is not placed too far away from the transmitters by moving it closer to the transmitters. • Check that the Wrist Receiver is set to the correct Radio Key. For further information see Function/Radio key.
The receivers in the system transmit signals for no reason.	• Change the Radio Key on all units in the system. For further information see Function/Radio key.

For further information about the product in English, refer to the Appendix.

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Further information

Settings

No adjustments are required for normal use. The relevant descriptions are provided below, if you wish to change a setting for some reason.

Radio key

In order to use several Bellman Visit 868 Systems close to one another without interference, different Radio Keys can be set on the different systems. All Bellman Visit 868 System units are supplied from the factory tuned to the same Radio Key, channel 0. This means that all Radio Key Switches on the transmitters are set to the OFF position.

A Bellman Visit 868 transmitter is also required to alter the Radio Key on the Bellman Visit 868 Wrist Receiver. Proceed as follows to change the Radio Key:

- Set a Bellman Visit 868 Transmitter to the desired Radio Key by altering its Radio Key Switch to the desired Radio Key. See the transmitter user manual for further information.
- Hold down the Function Button (1) on the Bellman Visit 868 Wrist

Receiver until the green  (3) and yellow  (4) LEDs blink alternately. Release the button and the Bellman Visit 868 Wrist Receiver will now be in programming mode for about 30 seconds.

- Press the Bellman Visit 868 transmitter's test button.
- The LEDs     (2, 3, 4 and 5) will now blink five times in quick succession to indicate a successful change of radio key.
- After changing the Radio Key, the Bellman Visit 868 Wrist Receiver will automatically return to normal mode.

Please note: all Bellman Visit 868 products within a system must be tuned to the same Radio Key in order to operate as a group.

Signal pattern

A Signal Pattern is the name for the way in which a receiver in the Bellman Visit 868 System indicates activation. Changing the transmitters' Signal Switch changes the Signal Pattern which the receivers display when the transmitter is activated.

Please note that the Wrist Receiver does not emit any sound or flash.

Signal Pattern for the Bellman Visit 868 System

Type	LED pattern	Sound	Vibration	Flash
Green 1	Green is constantly lit	1 x ding dong, low-frequency tone	Separate	Yes
Green 2	Green blinks in sequences of two	2 x ding dong, low-frequency tone	Separate	Yes
Green 3	Green blinks in sequences of three	1 x ding dong, high-frequency tone	Separate	Yes
Green 4	Green blinks constantly	2 x ding dong, high-frequency tone	Separate	Yes
Yellow 1	Yellow is constantly lit	1 x ring, low-frequency tone	Short	Yes
Yellow 2	Yellow blinks in sequences of two	2 x ring ring, low-frequency tone	Short	Yes
Yellow 3	Yellow blinks in sequences of three	1 x ring, high-frequency tone	Short	Yes
Yellow 4	Yellow blinks constantly	2 x ring ring, high-frequency tone	Short	Yes
Orange 1	Orange is constantly lit	Baby	Rapid	Yes
Orange 2	Orange blinks in sequences of two	Baby	Rapid	Yes
Orange 3	Orange blinks in sequences of three	Baby	Rapid	Yes
Orange 4	Orange blinks constantly	Baby	Rapid	Yes
VMA	Red and Orange constantly blink alternately	VMA constant	Long	Yes
Fire alarm	Red blinks constantly	Fire alarm constant	Long	Yes
Fire alarm low battery	Red blinks every five seconds	No	No	No
Fire alarm flat battery	Red blinks every five seconds	Fire alarm one short	One short	Yes

Please note that the Wrist Receiver does not emit any sound or flash.

Advanced programming

Advanced programming provides additional options for those who wish to make special modifications to the Bellman Visit 868 Wrist Receiver.

The idea is that it should be possible to select a completely unique signal pattern which is linked to activation from a specific input on a specific Bellman Visit 868 transmitter. The function works regardless of the Radio Key settings on the units that are programmed. Please note that, for safety reasons, the function does not work with the BE1480 Bellman Visit 868 Smoke Alarm.

By using advanced programming of the Bellman Visit 868 Wrist Receiver, it can be adjusted so that its signal pattern corresponds exactly to what is required. In other words, an entirely individual signal pattern can be programmed, such as displaying a permanently lit orange LED and a constant vibration.

In order to adjust the setting, the Bellman Visit 868 Transmitter to which the Bellman Visit 868 Wrist Receiver should be adapted must be available. The transmitter must also be connected so that it can be activated in the way in which it is intended to be used.

Proceed as follows:

1. Hold down the Function Button (1) on the Bellman Visit 868 Wrist Receiver until the green  (3) and yellow  (4) LEDs blink alternately. The Bellman Visit 868 Wrist Receiver will now be in programming mode for about 30 seconds.
2. Hold down the Function Button (1) at the same time as the relevant Bellman Visit 868 transmitter is activated in precisely the way in which it is intended to be used. The yellow LED  (6) will light up. **Note** that all inputs are individual, so that it is not possible to use the test button on a Bellman Visit 868 Telephone Transmitter whose telephone input will indicate the relevant pattern.
3. Scroll through the different LED options by a short press on the Function Button (1). Select the relevant indication by holding down the Function Button (1) until the LED  (6) goes out and starts to shine with a constant yellow light again.
4. Scroll through the different vibration options by a short press on the Function Button (1). Select the relevant indication by holding down the Function Button (1) until the LED  (6) goes out and starts to shine with a constant yellow light again.
5. The Bellman Visit 868 Wrist Receiver will now show the indication method programmed. End the display with a short press of the Function Button (1).

6. After a short while the Bellman Visit 868 Wrist Receiver will automatically return to normal mode.

This function is essential where a transmitter has to work in a special way with regard to a specific receiver.

Resetting advanced programming

It is quite easy to reset the Bellman Visit 868 Wrist Receiver if it needs to be reset after it has been programmed using advanced programming.

1. Hold down the Function Button (1) on the Bellman Visit 868 Wrist Receiver until the green  (3) and yellow  (4) LEDs blink alternately. The Bellman Visit 868 Wrist Receiver will now be in programming mode for about 30 seconds.
2. Press the Function Button (1) three times in quick succession.
3. All LEDs     (2-5) remain constantly on for a few seconds.
4. All the advanced programming has now been deleted and the Bellman Visit 868 Wrist Receiver will automatically return to normal mode.

Test

It is easy to test the BE1560 Bellman Visit 868 Wrist Receiver. If the Wrist Receiver does not work as described below, you can check further below under Appendix/Further information/Troubleshooting/Troubleshooting guide.

How to test

A transmitter in the Bellman Visit 868 System which is set to the same Radio Key as the Wrist Receiver is required to test the radio reception on the BE1560 Bellman Visit 868 Wrist Receiver.

- Press the transmitter test button.
- The Bellman Visit 868 Wrist Receiver will give the following indications:
 - o The LEDs     (2-5) which the transmitter has been set to indicate with or which have been programmed into the Bellman Visit 868 Wrist Receiver with Advanced Programming will be switched on.
 - o It will vibrate as the transmitter is set to indicate or in the way the Bellman Visit 868 Wrist Receiver has been programmed with Advanced Programming.

Troubleshooting

You can carry out a number of checks yourself before sending a product for repair.

Troubleshooting guide

<i>Problem</i>	<i>Solution</i>
Nothing happens.	• Check that the power supply unit is connected correctly and that the green LED (10) on the Charger lights up. • Charge or renew the battery in the Wrist Receiver. Only use a rechargeable VARTA V40H NiMH battery. If the battery is completely flat, you can still use the Wrist Receiver by placing it in the Charger. The Wrist Receiver needs to be charged for about two hours before it can be used separately. • Check that the Wrist Receiver has been correctly placed in the Wrist Receiver Charger and that the LED (12) lights up green.
It is not possible to charge the Wrist Receiver.	• Check that the power supply unit is connected correctly and that the green LEDs (10 and 12) on the Charger light up • Charge or change the batteries in the Wrist Receiver Charger. Only use rechargeable TMK AAA60CT NiMH batteries.

Problem	Solution
Nothing happens when the power supply unit is disconnected. Nor does the Test Button (11) work.	• Charge or change the batteries. Only use rechargeable TMK AAA60CT NiMH batteries.
The LED  (6) on the Wrist Receiver blinks yellow!	• Charge or renew the battery in the Wrist Receiver. Only use a rechargeable VARTA V40H NiMH battery. If the battery is completely flat, you can still use the Wrist Receiver by placing it in the Charger. The Wrist Receiver needs to be charged for about two hours before it can be used separately.
The Bed-shaker does not vibrate when the Wrist Receiver is activated.	• Check that the bed-shaker is correctly connected to the Wrist Receiver Charger. • Check that the Wrist Receiver has been correctly inserted in the Wrist Receiver Charger and that the LED (12) lights up green.
The Wrist Receiver does not pick up signals from the Bellman Visit 868 Transmitters.	• Check the batteries in the transmitters. • Check that the Wrist Receiver is not placed too far away by moving it closer to the transmitter. • Check that the Bellman Visit 868 Wrist Receiver is set to the same Radio Key as the other units in the relevant Bellman Visit 868 System. For further information see Appendix/ Further information/Settings/Radio key.
The Wrist Receiver is emitting the wrong signals, regardless of how the transmitter is set.	• Delete the advanced programming, see Appendix/Further information/Resetting advanced programming.
The receiver signals when no transmitter is activated.	• Change the Radio Key on all units in the relevant Bellman Visit 868 System. There is probably another system nearby with the same Radio Key.

Technical information BE1560

Power supply

Mains power:

Via Bellman Visit Wrist Receiver Charger, BE1570.

Battery power:

1 x 1.2 V VARTA V40H rechargeable NiMH battery. The battery cover can be opened with a coin.

Please note! ONLY USE RECHARGEABLE VARTA V40H NiMH BATTERIES in Bellman Visit 868 Wrist Receiver, BE1560!

Non-rechargeable batteries will start to leak if recharged! The acid in the batteries will damage the electronics in the BE 1560 Bellman Visit 868 Wrist Receiver and the resulting damage will not be covered by any guarantee.

Operating time:

Approximately 30 hours

Charging time:

Approximately 8 hours. If the battery is completely flat, you can still use the Wrist Receiver by placing it in the Charger. The Wrist Receiver needs to be charged for about two hours before it can be used separately.

Power consumption:

Active: Approximately 100 mA

Idle position: Approximately 3 mA

Radio function

Radio frequency:

868.3 MHz

Number of Radio Keys:

64 Radio Keys as standard. Special software can be used to increase this to 256 Radio Keys in increments of 64 per software purchase. Contact the nearest supplier for further information.

Coverage:

The normal coverage between a transmitter and Bellman Visit 868 Wrist Receiver is approximately 100 metres with a clear line of sight. Coverage is reduced if walls and large objects screen off the signal. Any thick walls constructed of reinforced concrete will greatly affect coverage.

The system may also be affected by radio transmitters such as TV transmitters, computers, mobile phones, etc. This means that a unit may work perfectly in one part of the room but not at all in another.

Activation**Radio:**

Bellman Visit 868 System

Output signals**Vibrator power:**

Built-in

Additional information

For indoor use only

Dimensions W x H x D:

38 x 12 x 49 mm

Weight:

With battery: 27 g

Without battery: 25 g

Colour:

White with black top part and grey Function Button.

Accessories

Bellman Visit Wrist Receiver Charger, BE1570

Bellman Bed-shaker, BE1270 (connected to BE1570)

Bellman Wrist Belt Tool, BE9140

Technical information BE1570

Power supply

Mains power:

7.5 V DC/1500 mA with power supply unit BE9201 (Europe) and BE9202 (United Kingdom).

Battery power:

4 x 1.2 V/600 mA AAA NiMH TMK AAA60CT rechargeable batteries.

Charging:

Via the power supply unit. Discharged back-up batteries will take at least 24 hours to charge. The BE1570 should normally be permanently connected to mains power.

Power consumption:

Active: 650 mA

Idle position: 70 mA

Please note! ONLY USE RECHARGEABLE VARTA TMK AAA60CT NiMH BATTERIES in the Bellman Visit 868 Wrist Receiver Charger, BE1570!

Non-rechargeable batteries will start to leak if recharged! The acid in the batteries will damage the electronics in the BE1570 Bellman Visit 868 Wrist Receiver Charger and the resulting damage will not be covered by any guarantee.

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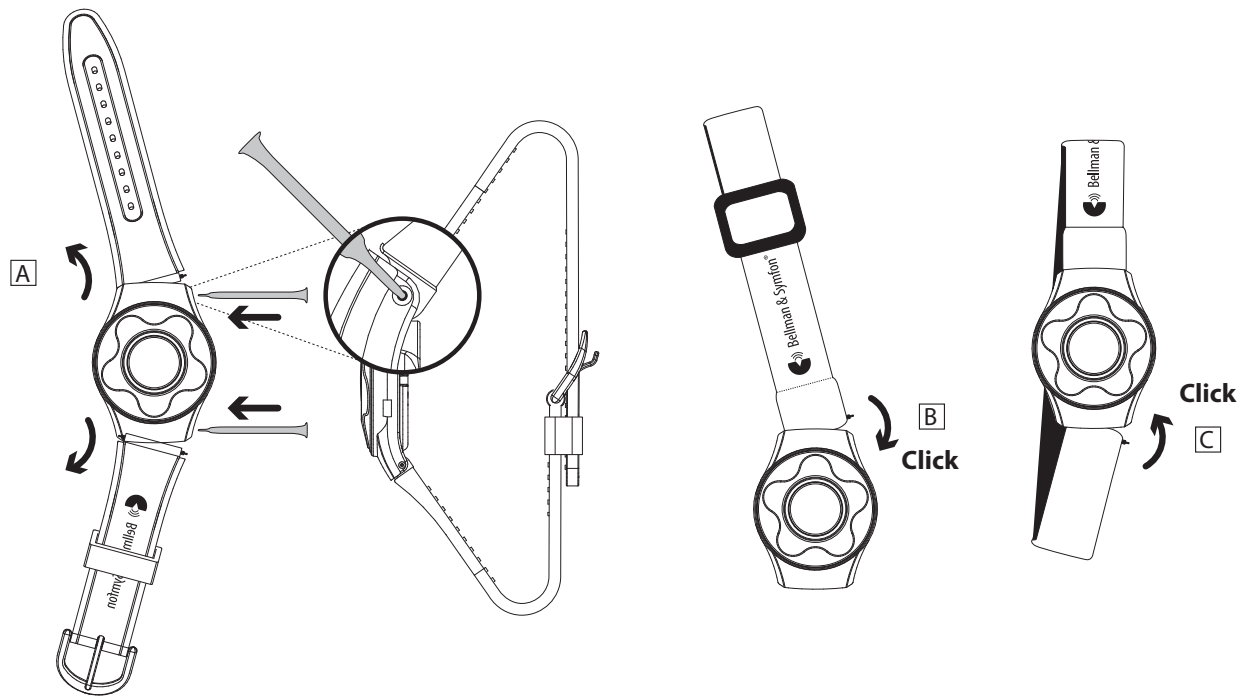
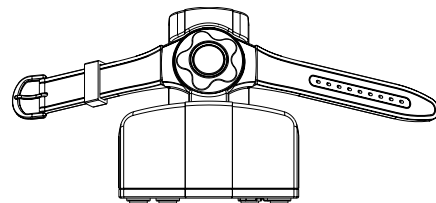
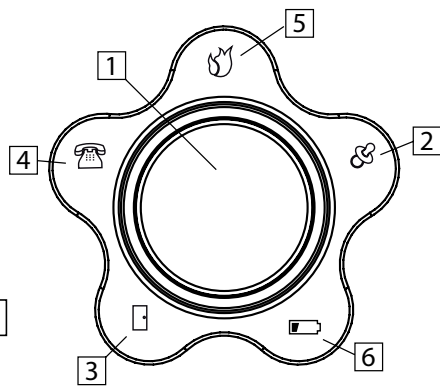
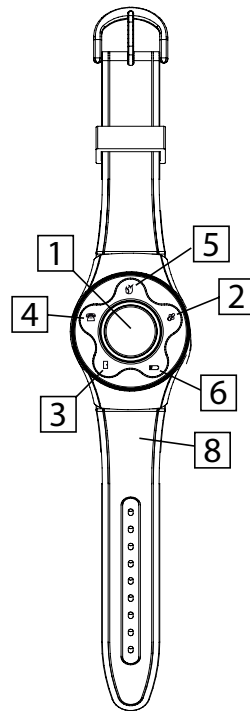
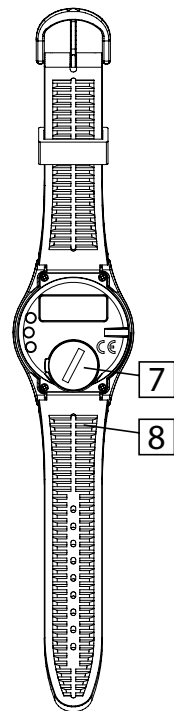
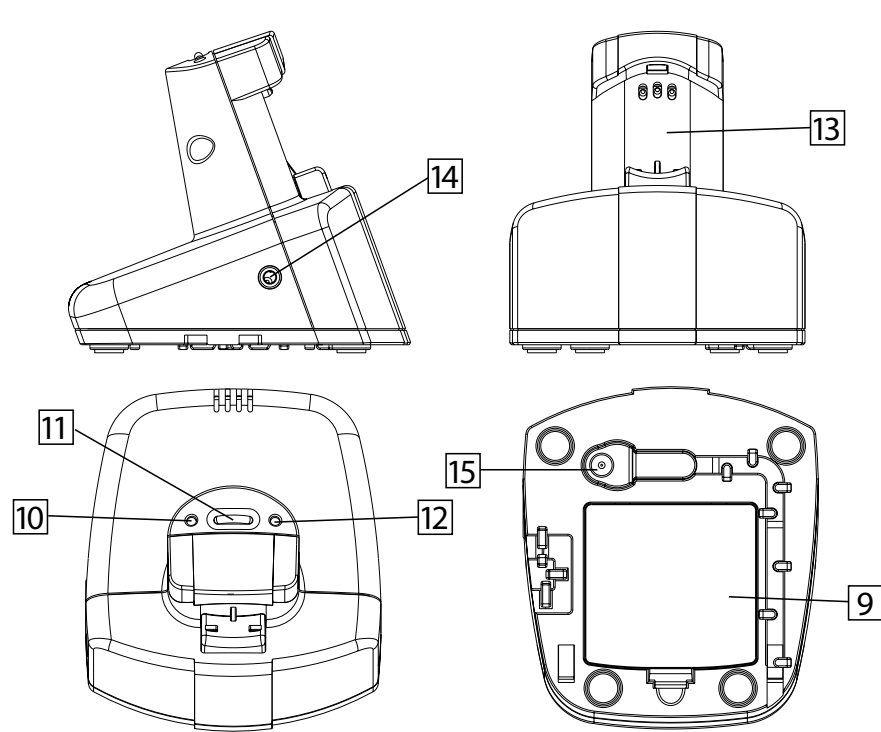
For the safety of your children

Please be aware that this product may include small parts. Ensure that the package and small parts are kept out of reach from your child.

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